

Enabling 'MultiUserMode' for IntelViewer on Windows Server 2012 R2

Revised Apr 30, 2014 by r.a. parks.

Overview:

The following is a companion PDF file linked from a blog post: [IntelViewer on Windows Server 2012 R2](#). The post's author has no affiliation with Intelrad Medical Systems Inc., other than that of a paying customer. Use at your own risk, as follows.

- Copy the content shown in gray below to Windows Notepad. If the line breaks disappear, re-insert them to mimic what you see here.
- Save it.
- Change the file extension to .REG.
- Double-click to execute it on your Windows 2012 Server.

```
Windows Registry Editor Version 5.00
```

```
[HKEY_LOCAL_MACHINE\SOFTWARE\Intelrad Medical Systems\IntelViewer]  
"MultiUserMode"="true"
```

```
[HKEY_LOCAL_MACHINE\SOFTWARE Wow6432Node\Intelrad Medical Systems\IntelViewer]  
"MultiUserMode"="true"
```

